



CONTROL	L	N	H
---------	---	---	---

**IMPORTANT:** The barcode is for use only on the CELL-DYN Ruby. Refer to the appropriate System Operator's Manual for proper use of CELL-DYN Calibrator and Control Products.


**IMPORTANT: Mixing and Handling**

1. Remove a vial of the control from the refrigerator and warm to room temperature (18° to 30° C) for 15 minutes before use.

2. To mix: (**Do NOT mix mechanically or vortex.**)

For a video demonstration, visit [www.corelaboratory.abbott](http://www.corelaboratory.abbott) and navigate to the Customer Portal → Technical Library → Other Reference Documents → Hematology Aids.

a. Hold the vial vertically and roll each vial between the palms of the hands for 15-20 seconds.

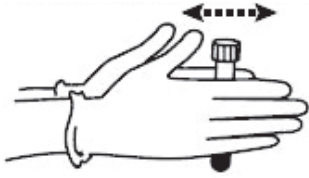

b. Continue to mix by holding the vial by the ends between the thumb and finger, rapidly inverting the vial 20 times end-over-end using a very quick turning motion of the wrist.



c. Analyze immediately after mixing. Subsequent analyses during this test period may be performed by inverting the vial 5 times prior to instrument analysis.

d. Steps a-c must be repeated upon removing the sample from the refrigerator for the entire open-vial time period regardless of the method of analysis (open tube, cap piercing, auto sample or manual sample).

3. Refer to the appropriate CELL-DYN System Operator's Manual for information about analyzing control specimens.

4. FOR AUTOMATED SAMPLING OR MANUAL CLOSED SAMPLING (CS):

- Refer to the appropriate CELL-DYN Operator's Manual. Remove the vial from the sample handler immediately after sampling.

FOR OPEN-VIAL SAMPLING:

- Aspirate a sample from the vial.
- Carefully wipe the vial rim and cap with a lint-free tissue.
- Replace the cap, ensuring it is on tight.

After sampling, return vial to refrigerator for maximum open-vial stability. If run in the open mode, wipe the threads of both vial and cap before replacing cap and returning to refrigerator.

# CELL-DYN 26 Plus Control

ABBOTT  
CELL-DYN SYSTEMS



ASSAY SHEET



2020-09-25

8 Consecutive Day Open-Vial Stability

		CONTROL L		CONTROL N		CONTROL H	
		LOT L0195		LOT N0195		LOT H0195	
SYSTEM	PARAMETER	ASSAY VALUE	± MEAN RANGE *	ASSAY VALUE	± MEAN RANGE *	ASSAY VALUE	± MEAN RANGE *
CELL-DYN 3700 SYSTEM	WOC 10 <sup>9</sup> /L	4.0	0.4	7.2	0.7	15.9	2.5
	WIC 10 <sup>9</sup> /L	4.0	0.5	7.1	1.0	15.9	3.0
	WBC 10 <sup>9</sup> /L	4.0	0.4	7.2	0.7	15.9	2.5
	NEU 10 <sup>9</sup> /L	2.3	0.3	4.2	0.8	9.3	2.0
	NEU %	57.9	8.0	57.9	8.0	58.4	10.0
	LYM 10 <sup>9</sup> /L	1.1	0.3	2.0	0.8	4.3	2.0
	LYM %	27.0	9.0	27.3	9.0	26.9	10.0
	MONO 10 <sup>9</sup> /L	0.4	0.2	0.7	0.4	1.6	0.6
	MONO %	10.3	5.0	9.9	5.0	9.8	3.0
	EOS 10 <sup>9</sup> /L	0.1	0.1	0.2	0.2	0.5	0.2
	EOS %	3.0	3.0	3.0	2.0	2.9	1.0
	BASO 10 <sup>9</sup> /L	0.1	0.1	0.2	0.2	0.6	0.6
	BASO %	3.0	3.0	3.0	3.0	3.0	3.0
	RBC 10 <sup>12</sup> /L	2.87	0.15	4.26	0.20	5.23	0.30
	HGB g/dL	7.3	0.4	11.9	0.6	15.8	0.8
	HCT %	22.8	1.5	36.9	2.5	48.4	3.5
	MCV fL	79.3	4.0	86.7	4.0	92.5	5.0
MCH pg	25.4	2.0	27.9	2.0	30.2	2.0	
MCHC g/dL	32.0	2.3	32.2	3.0	32.6	2.3	
RDW %	19.4	2.5	18.5	2.5	18.1	2.5	
PLT 10 <sup>9</sup> /L	74	20	222	30	545	60	
MPV fL	8.4	3.0	7.9	3.0	7.6	3.0	
CELL-DYN Ruby SYSTEM	WBC (WOC) 10 <sup>9</sup> /L	3.9	0.4	7.2	0.7	16.0	2.5
	WBC (NOC) 10 <sup>9</sup> /L	4.0	0.4	7.3	1.0	16.3	2.5
	NEU 10 <sup>9</sup> /L	2.3	0.3	4.2	0.8	9.4	2.0
	NEU %	57.7	8.0	58.1	8.0	58.5	10.0
	LYM 10 <sup>9</sup> /L	1.0	0.3	1.9	0.8	4.2	2.0
	LYM %	26.2	9.0	26.8	9.0	26.5	10.0
	MONO 10 <sup>9</sup> /L	0.4	0.2	0.7	0.4	1.5	0.6
	MONO %	10.0	5.0	9.6	5.0	9.5	3.0
	EOS 10 <sup>9</sup> /L	0.1	0.1	0.2	0.2	0.5	0.2
	EOS %	3.1	3.0	2.9	2.0	2.9	1.0
	BASO 10 <sup>9</sup> /L	0.1	0.1	0.2	0.2	0.6	0.6
	BASO %	3.0	3.0	3.0	3.0	3.0	3.0
	RBC 10 <sup>12</sup> /L	2.80	0.15	4.26	0.20	5.31	0.30
	HGB g/dL	7.2	0.4	11.8	0.6	15.8	0.8
	HCT %	19.7	1.5	32.0	2.5	41.6	3.5
	MCV fL	70.3	4.0	75.0	4.0	78.3	5.0
	MCH pg	25.7	2.0	27.7	2.0	29.8	2.0
MCHC g/dL	36.5	2.3	36.9	3.0	38.0	2.3	
RDW %	13.8	2.5	12.6	2.5	12.0	2.5	
PLT 10 <sup>9</sup> /L	69	20	206	30	509	60	
MPV fL	10.9	3.0	10.8	3.0	10.9	3.0	

\* The **MEAN RANGE** does not represent standard deviations (SD).  
NOTE: Flags may occur with control materials and should be disregarded.



CELL-DYN and CELL-DYN Ruby are trademarks of Abbott Laboratories in various jurisdictions.



Abbott Laboratories  
Diagnostics Division  
Abbott Park, IL 60064  
USA



Abbott GmbH & Co. KG  
Max-Planck-Ring 2  
65205 Wiesbaden  
Germany  
+49-6122-580

**MANUFACTURED FOR**  
Abbott Laboratories



**REF** 08H59-01, 08H59-02



9231592B 350004-6 August 2018  
©2017,2018 Abbott Laboratories