

CELL-DYN 29 Plus Control (with Retic)

ABBOTT
CELL-DYN SYSTEMS



ASSAY SHEET

CONTROL | L | N | H


Refer to the appropriate system operator's manual for proper use of CELL-DYN Calibrator and Control Products.

IMPORTANT: Mixing and Handling

1. Remove a vial of the control from the refrigerator and warm to room temperature (18° to 30° C) for 15 minutes before use.
2. To mix: **(Do NOT mix mechanically or vortex.)**

For a video demonstration, visit www.corelaboratory.abbott and navigate to the Customer Portal → Technical Library → Other Reference Documents → Hematology Aids.

- a. Hold the vial vertically and roll each vial between the palms of the hands for 15-20 seconds.

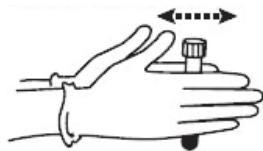

- b. Continue to mix by holding the vial by the ends between the thumb and finger, rapidly inverting the vial 20 times end-over-end using a very quick turning motion of the wrist.



- c. Analyze immediately after mixing. Subsequent analyses during this test period may be performed by inverting the vial 5 times prior to instrument analysis.
- d. Steps a-c must be repeated upon removing the sample from the refrigerator for the entire open-vial time period regardless of the method of analysis (open tube, cap piercing, auto sample or manual sample).

3. Refer to the appropriate CELL-DYN System Operator's Manual for information about analyzing control specimens.

NOTE: For CELL-DYN 3700 and CELL-DYN Ruby:

- Perform stain of CELL-DYN 29 Plus Control (with Retic) as a patient sample as described in the CELL-DYN 3700 and CELL-DYN Ruby Reticulocyte Reagent package insert, except limit the staining time to between 15 and 30 minutes.

4. FOR AUTOMATED SAMPLING OR MANUAL CLOSED SAMPLING (CS):

- Refer to the appropriate CELL-DYN Operator's Manual. Remove the vial from the sample handler immediately after sampling.

FOR OPEN-VIAL SAMPLING:

- Aspirate a sample from the vial.
- Carefully wipe the vial rim and cap with a lint-free tissue.
- Replace the cap, ensuring it is on tight.

After sampling, return vial to refrigerator for maximum open-vial stability. If run in the open mode, wipe the threads of both vial and cap before replacing cap and returning to refrigerator.



Exp. 2020-09-25

7 Consecutive Day Open-Vial Stability

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
CELL-DYN Sapphire	LOT L01959		LOT N01959		LOT H01959	
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
WBC 10 ⁹ /L	2.98	0.40	6.98	0.80	16.3	3.0
NEU 10 ⁹ /L	1.69	0.20	4.07	0.40	10.4	1.1
NEU %	56.8	6.0	58.3	5.0	64.1	6.0
LYM 10 ⁹ /L	0.91	0.20	1.92	0.50	3.46	1.00
LYM %	30.5	8.0	27.5	6.0	21.2	5.0
MONO 10 ⁹ /L	0.30	0.20	0.77	0.40	1.92	0.60
MONO %	10.1	6.0	11.1	5.0	11.8	3.0
EOS 10 ⁹ /L	0.10	0.10	0.20	0.17	0.45	0.19
EOS %	3.00	3.00	2.90	2.00	2.75	1.00
BASO 10 ⁹ /L	0.10	0.10	0.25	0.25	0.50	0.50
BASO %	1.50	1.50	1.50	1.50	1.50	1.50
RBC 10 ¹² /L	2.81	0.18	4.19	0.20	5.24	0.30
RBCo 10 ¹² /L	2.87	0.18	4.20	0.20	5.21	0.30
HGB g/dL	7.45	0.30	11.5	0.5	16.3	0.8
HCT %	22.3	1.5	33.7	2.5	46.9	3.0
MCV fL	79.4	4.0	80.5	4.0	89.5	4.0
MCH pg	26.5	2.0	27.4	2.0	31.1	2.0
MCHC g/dL	33.4	2.3	34.1	2.3	34.8	2.3
RDW %	15.8	2.5	15.4	2.5	14.4	2.5
NRBC 10 ⁹ /L*	0.001	0.001	0.001	0.001	2.33	0.60
NRBC/100WBC*	0.001	0.001	0.001	0.001	14.3	2.5
PLT 10 ⁹ /L	69.5	20.0	205	50	443	60
PLTi 10 ⁹ /L	74.3	20.0	224	50	466	60
MPV fL	9.19	2.00	8.25	2.00	7.73	2.00
RETC 10 ⁹ /L	221	50	140	50	94.8	50.0
%R	7.85	1.50	3.33	1.00	1.81	0.80
IRF	0.64	0.18	0.53	0.14	0.38	0.10



Exp. 2020-09-25

7 Consecutive Day Open-Vial Stability

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
CELL-DYN Ruby	LOT L01959		LOT N01959		LOT H01959	
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
WOC 10 ⁹ /L	3.0	0.4	7.0	0.7	16.1	2.5
NOC 10 ⁹ /L	3.1	0.4	7.1	1.0	18.7	2.5
NEU 10 ⁹ /L	1.7	0.3	4.1	0.8	10.4	2.0
NEU %	56.5	6.0	58.5	6.0	64.4	10.0
LYM 10 ⁹ /L	0.9	0.3	1.8	0.8	3.2	2.0
LYM %	28.9	7.0	26.3	6.0	20.0	10.0
MONO 10 ⁹ /L	0.3	0.2	0.7	0.4	1.7	0.6
MONO %	9.7	5.0	10.3	4.5	10.6	3.0
EOS 10 ⁹ /L	0.1	0.1	0.2	0.2	0.5	0.2
EOS %	3.0	3.0	2.8	2.0	2.8	1.0
BASO 10 ⁹ /L	0.1	0.1	0.2	0.2	0.6	0.6
BASO %	3.0	3.0	3.0	3.0	3.0	3.0
RBC 10 ¹² /L	2.71	0.15	4.09	0.20	5.21	0.28
HGB g/dL	7.2	0.4	11.4	0.5	16.7	0.6
HCT %	20.0	1.5	30.8	2.3	43.1	3.5
MCV fL	73.8	4.0	75.3	4.0	82.8	4.0
MCH pg	26.6	2.0	27.9	2.0	32.1	2.0
MCHC g/dL	36.0	2.3	37.0	3.0	38.7	2.3
RDW %	13.7	2.5	12.5	2.5	11.0	2.5
PLT 10 ⁹ /L	71	20	213	30	493	60
MPV fL	6.8	2.0	6.2	2.0	6.3	2.0
Retic %***	6.7	1.5	2.9	1.0	1.3	0.8

NOTE: Flags may occur with control materials. PIC/POC alarms may be seen with this control when used on the CELL-DYN Sapphire. The alarms may be disregarded if the control is performing within the assay ranges.

* NOTE: The Assay Value of .001 and Mean Range of ± .001 for NRBC and NRBC/100WBC is entered for the Level L and Level N controls since the instrument will not accept a value of zero. The NRBC concentration for Levels L and N is below the detectable level of the instrument and as such serves as the NRBC negative control. The Level H is the NRBC positive control.

** The mean range does not represent standard deviations (SD).

*** Retic % values for CELL-DYN Ruby are included as separate files on assay disk.

CELL-DYN 29 Plus Control (with Retic)

CONTROL L N H

 Exp. 2020-09-25	7 Consecutive Day Open-Vial Stability
---	---------------------------------------

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
	LOT	L01959	LOT	N01959	LOT	H01959
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
CELL-DYN 3700						
WOC 10 ⁹ /L	3.1	0.4	7.0	0.7	16.1	2.5
WIC 10 ⁹ /L	3.1	0.5	7.3	1.0	19.3	3.0
WBC 10 ⁹ /L	3.1	0.4	7.0	0.7	16.1	2.5
NEU 10 ⁹ /L	1.8	0.3	4.1	0.8	10.4	2.0
NEU %	56.7	6.0	58.1	6.0	64.4	10.0
LYM 10 ⁹ /L	0.9	0.3	1.8	0.8	3.2	2.0
LYM %	29.1	7.0	26.3	6.0	19.8	10.0
MONO 10 ⁹ /L	0.3	0.2	0.8	0.4	1.8	0.6
MONO %	10.1	5.0	11.0	4.5	11.3	3.0
EOS 10 ⁹ /L	0.1	0.1	0.2	0.2	0.5	0.2
EOS %	3.0	3.0	2.9	2.0	2.8	1.0
BASO 10 ⁹ /L	0.1	0.1	0.3	0.3	0.6	0.6
BASO %	3.0	3.0	3.0	3.0	3.0	3.0
RBC 10 ¹² /L	2.83	0.15	4.18	0.20	5.29	0.28
HGB g/dL	7.4	0.3	11.7	0.5	17.0	0.6
HCT %	23.6	1.5	36.3	2.3	51.8	3.5
MCV fL	83.5	4.0	86.9	4.0	98.0	4.0
MCH pg	26.1	2.0	28.0	2.0	32.1	2.0
MCHC g/dL	31.4	2.3	32.2	3.0	32.8	2.3
RDW %	19.1	2.5	19.1	2.5	17.0	2.5
PLT 10 ⁹ /L	70	20	217	30	467	60
MPV fL	6.9	2.0	6.8	2.0	7.0	2.0
Retic % ¹	6.3	1.5	2.9	1.0	1.1	0.8
IRF ²	0.62	0.38	0.57	0.30	0.39	0.20

 Exp. 2020-09-25	7 Consecutive Day Open-Vial Stability
---	---------------------------------------

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
	LOT	L01959	LOT	N01959	LOT	H01959
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
Manual Count ³						
Retic %	5.6	2.0	2.6	1.5	0.8	0.8

** The mean range does not represent standard deviations (SD).


¹ Retic % values will not load from the Assay Disk. Please enter these values manually.

² IRF is reportable on the CELL-DYN 3700 System, Version 1.1 and higher.

³ Manual values were obtained using the Miller Ocular method.



CELL-DYN, CELL-DYN Sapphire and CELL-DYN Ruby are trademarks of Abbott Laboratories in various jurisdictions.

 Abbott Laboratories
Diagnostics Division
Abbott Park, IL 60064
USA

EC REP Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

MANUFACTURED FOR
Abbott Laboratories



REF 08H58-01, 08H58-02



9231566B 350491-10 August 2018
©2017, 2018 Abbott Laboratories