



CONTROL	L	N	H
---------	---	---	---

IMPORTANT: The barcode is for use only on the CELL-DYN Ruby. Refer to the appropriate System Operator's Manual for proper use of CELL-DYN Calibrator and Control Products.


IMPORTANT: Mixing and Handling

1. Remove a vial of the control from the refrigerator and warm to room temperature (18° to 30° C) for 15 minutes before use.

2. To mix: (**Do NOT mix mechanically or vortex.**)

For a video demonstration, visit www.corelaboratory.abbott and navigate to the Customer Portal → Technical Library → Other Reference Documents → Hematology Aids.

a. Hold the vial vertically and roll each vial between the palms of the hands for 15-20 seconds.

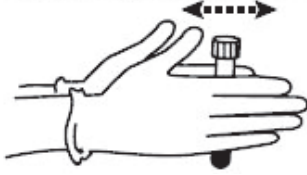

b. Continue to mix by holding the vial by the ends between the thumb and finger, rapidly inverting the vial 20 times end-over-end using a very quick turning motion of the wrist.



c. Analyze immediately after mixing. Subsequent analyses during this test period may be performed by inverting the vial 5 times prior to instrument analysis.

d. Steps a-c must be repeated upon removing the sample from the refrigerator for the entire open-vial time period regardless of the method of analysis (open tube, cap piercing, auto sample or manual sample).

3. Refer to the appropriate CELL-DYN System Operator's Manual for information about analyzing control specimens.

4. FOR AUTOMATED SAMPLING OR MANUAL CLOSED SAMPLING (CS):

- Refer to the appropriate CELL-DYN Operator's Manual. Remove the vial from the sample handler immediately after sampling.

FOR OPEN-VIAL SAMPLING:

- Aspirate a sample from the vial.
- Carefully wipe the vial rim and cap with a lint-free tissue.
- Replace the cap, ensuring it is on tight.

After sampling, return vial to refrigerator for maximum open-vial stability. If run in the open mode, wipe the threads of both vial and cap before replacing cap and returning to refrigerator.



2022-03-11

8 Consecutive Day Open-Vial Stability

		CONTROL L		CONTROL N		CONTROL H	
		LOT L1361		LOT N1361		LOT H1361	
SYSTEM	PARAMETER	ASSAY VALUE	± MEAN RANGE *	ASSAY VALUE	± MEAN RANGE *	ASSAY VALUE	± MEAN RANGE *
CELL-DYN 3700 SYSTEM	WOC 10 ⁹ /L	3.5	0.4	6.7	0.7	15.6	2.5
	WIC 10 ⁹ /L	3.7	0.5	6.9	1.0	15.8	3.0
	WBC 10 ⁹ /L	3.5	0.4	6.7	0.7	15.6	2.5
	NEU 10 ⁹ /L	2.1	0.3	3.9	0.8	9.2	2.0
	NEU %	58.6	8.0	58.7	8.0	59.1	10.0
	LYM 10 ⁹ /L	0.9	0.3	1.8	0.8	4.1	2.0
	LYM %	26.7	9.0	26.6	9.0	26.1	10.0
	MONO 10 ⁹ /L	0.4	0.2	0.6	0.4	1.5	0.6
	MONO %	10.0	5.0	9.7	5.0	9.6	3.0
	EOS 10 ⁹ /L	0.1	0.1	0.2	0.2	0.5	0.2
	EOS %	3.0	3.0	2.8	2.0	2.9	1.0
	BASO 10 ⁹ /L	0.1	0.1	0.2	0.2	0.6	0.6
	BASO %	3.0	3.0	3.0	3.0	3.0	3.0
	RBC 10 ¹² /L	2.96	0.15	4.25	0.20	5.25	0.30
	HGB g/dL	6.9	0.4	12.1	0.6	16.2	0.8
	HCT %	21.4	1.5	37.4	2.5	49.0	3.5
	MCV fL	72.4	4.0	87.9	4.0	93.3	5.0
MCH pg	23.3	2.0	28.5	2.0	30.9	2.0	
MCHC g/dL	32.2	2.3	32.4	3.0	33.1	2.3	
RDW %	21.8	2.5	18.2	2.5	17.8	2.5	
PLT 10 ⁹ /L	77	20	222	30	549	60	
MPV fL	8.4	3.0	7.7	3.0	7.4	3.0	
CELL-DYN Ruby SYSTEM	WBC (WOC) 10 ⁹ /L	3.4	0.4	6.9	0.7	16.2	2.5
	WBC (NOC) 10 ⁹ /L	3.5	0.4	7.0	1.0	16.2	2.5
	NEU 10 ⁹ /L	2.0	0.3	4.1	0.8	9.6	2.0
	NEU %	58.8	8.0	58.7	8.0	59.5	10.0
	LYM 10 ⁹ /L	0.9	0.3	1.8	0.8	4.2	2.0
	LYM %	25.7	9.0	25.8	9.0	25.7	10.0
	MONO 10 ⁹ /L	0.3	0.2	0.7	0.4	1.5	0.6
	MONO %	9.2	5.0	9.6	5.0	9.0	3.0
	EOS 10 ⁹ /L	0.1	0.1	0.2	0.2	0.5	0.2
	EOS %	3.0	3.0	2.8	2.0	2.8	1.0
	BASO 10 ⁹ /L	0.1	0.1	0.2	0.2	0.6	0.6
	BASO %	3.3	3.0	3.1	3.0	3.0	3.0
	RBC 10 ¹² /L	2.94	0.15	4.31	0.20	5.34	0.30
	HGB g/dL	6.7	0.4	12.0	0.6	16.2	0.8
	HCT %	19.3	1.5	32.7	2.5	42.3	3.5
	MCV fL	65.8	4.0	75.8	4.0	79.3	5.0
	MCH pg	22.8	2.0	27.8	2.0	30.3	2.0
MCHC g/dL	34.7	2.3	36.7	3.0	38.3	2.3	
RDW %	15.3	2.5	12.9	2.5	12.1	2.5	
PLT 10 ⁹ /L	73	20	221	30	561	60	
MPV fL	10.1	3.0	9.9	3.0	9.9	3.0	

* The **MEAN RANGE** does not represent standard deviations (SD).
NOTE: Flags may occur with control materials and should be disregarded.



CELL-DYN and CELL-DYN Ruby are trademarks of Abbott Laboratories in various jurisdictions.

Abbott Laboratories
Diagnostics Division
Abbott Park, IL 60064
USA

EC REP Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

MANUFACTURED FOR
Abbott Laboratories



REF 08H59-01, 08H59-02

