

CELL-DYN 29 Plus Control (with Retic)

ABBOTT
CELL-DYN SYSTEMS



CONTROL | L | N | H


Refer to the appropriate system operator's manual for proper use of CELL-DYN Calibrator and Control Products.

IMPORTANT: Mixing and Handling

1. Remove a vial of the control from the refrigerator and warm to room temperature (18° to 30° C) for 15 minutes before use.
2. To mix: (**Do NOT mix mechanically or vortex.**)

For a video demonstration, visit www.corelaboratory.abbott and navigate to the Customer Portal → Technical Library → Other Reference Documents → Hematology Aids.

- a. Hold the vial vertically and roll each vial between the palms of the hands for 15-20 seconds.

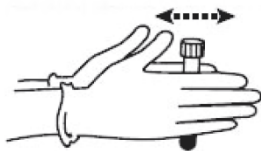

- b. Continue to mix by holding the vial by the ends between the thumb and finger, rapidly inverting the vial 20 times end-over-end using a very quick turning motion of the wrist.



- c. Analyze immediately after mixing. Subsequent analyses during this test period may be performed by inverting the vial 5 times prior to instrument analysis.
- d. Steps a-c must be repeated upon removing the sample from the refrigerator for the entire open-vial time period regardless of the method of analysis (open tube, cap piercing, auto sample or manual sample).

3. Refer to the appropriate CELL-DYN System Operator's Manual for information about analyzing control specimens.

NOTE: For CELL-DYN 3700 and CELL-DYN Ruby:

- Perform stain of CELL-DYN 29 Plus Control (with Retic) as a patient sample as described in the CELL-DYN 3700 and CELL-DYN Ruby Reticulocyte Reagent package insert, except limit the staining time to between 15 and 30 minutes.

4. FOR AUTOMATED SAMPLING OR MANUAL CLOSED SAMPLING (CS):

- Refer to the appropriate CELL-DYN Operator's Manual. Remove the vial from the sample handler immediately after sampling.

FOR OPEN-VIAL SAMPLING:

- Aspirate a sample from the vial.
- Carefully wipe the vial rim and cap with a lint-free tissue.
- Replace the cap, ensuring it is on tight.

After sampling, return vial to refrigerator for maximum open-vial stability. If run in the open mode, wipe the threads of both vial and cap before replacing cap and returning to refrigerator.



Exp. 2022-02-11

7 Consecutive Day Open-Vial Stability



Exp. 2022-02-11

7 Consecutive Day Open-Vial Stability

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
	LOT	L13339	LOT	N13339	LOT	H13339
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
WBC 10 ⁹ /L	3.08	0.40	7.06	0.80	16.3	3.0
NEU 10 ⁹ /L	1.80	0.20	4.32	0.40	10.7	1.1
NEU %	58.5	6.0	61.2	5.0	65.3	6.0
LYM 10 ⁹ /L	0.94	0.20	1.77	0.50	3.45	1.00
LYM %	30.6	8.0	25.1	6.0	21.1	5.0
MONO 10 ⁹ /L	0.25	0.20	0.74	0.40	1.64	0.60
MONO %	8.13	6.00	10.5	5.0	10.0	3.0
EOS 10 ⁹ /L	0.10	0.10	0.20	0.17	0.55	0.19
EOS %	3.00	3.00	2.89	2.00	3.38	1.00
BASO 10 ⁹ /L	0.10	0.10	0.25	0.25	0.50	0.50
BASO %	1.50	1.50	1.50	1.50	1.50	1.50
RBC 10 ¹² /L	2.90	0.18	4.26	0.20	5.27	0.30
RBC _o 10 ¹² /L	3.00	0.18	4.32	0.20	5.29	0.30
HGB g/dL	7.76	0.30	12.5	0.5	16.5	0.8
HCT %	22.3	1.5	35.2	2.5	45.9	3.0
MCV fL	76.7	4.0	82.8	4.0	87.2	4.0
MCH pg	26.7	2.0	29.3	2.0	31.3	2.0
MCHC g/dL	34.8	2.3	35.4	2.3	35.9	2.3
RDW %	15.6	2.5	14.9	2.5	13.2	2.5
NRBC 10 ⁹ /L*	0.001	0.001	0.001	0.001	2.29	0.60
NRBC/100WBC*	0.001	0.001	0.001	0.001	14.0	2.5
PLT 10 ⁹ /L	73.8	20.0	221	50	484	60
PLTi 10 ⁹ /L	79.5	20.0	230	50	490	60
MPV fL	10.0	2.0	7.54	2.00	7.13	2.00
RETC 10 ⁹ /L	229	50	143	50	90.2	50.0
%R	7.90	1.50	3.35	1.00	1.71	0.80
IRF	0.50	0.18	0.42	0.14	0.36	0.10

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
	LOT	L13339	LOT	N13339	LOT	H13339
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
WOC 10 ⁹ /L	3.2	0.4	7.3	0.7	16.7	2.5
NOC 10 ⁹ /L	3.1	0.4	7.2	1.0	18.7	2.5
NEU 10 ⁹ /L	1.8	0.3	4.5	0.8	10.9	2.0
NEU %	55.7	6.0	61.3	6.0	65.4	10.0
LYM 10 ⁹ /L	0.9	0.3	1.8	0.8	3.3	2.0
LYM %	29.3	7.0	24.1	6.0	19.7	10.0
MONO 10 ⁹ /L	0.3	0.2	0.7	0.4	1.6	0.6
MONO %	9.7	5.0	9.4	4.5	9.3	3.0
EOS 10 ⁹ /L	0.1	0.1	0.2	0.2	0.6	0.2
EOS %	3.0	3.0	2.7	2.0	3.4	1.0
BASO 10 ⁹ /L	0.1	0.1	0.2	0.2	0.6	0.6
BASO %	3.0	3.0	3.0	3.0	3.0	3.0
RBC 10 ¹² /L	2.91	0.15	4.31	0.20	5.37	0.28
HGB g/dL	7.6	0.4	12.6	0.5	17.1	0.6
HCT %	20.9	1.5	33.2	2.3	43.5	3.5
MCV fL	72.0	4.0	77.0	4.0	81.0	4.0
MCH pg	26.1	2.0	29.2	2.0	31.9	2.0
MCHC g/dL	36.2	2.3	38.0	3.0	39.3	2.3
RDW %	13.5	2.5	12.2	2.5	10.3	2.5
PLT 10 ⁹ /L	77	20	237	30	529	60
MPV fL	6.3	2.0	5.8	2.0	5.8	2.0
Retic %***	5.7	1.5	2.2	1.0	1.0	0.8

NOTE: Flags may occur with control materials. PIC/POC alarms may be seen with this control when used on the CELL-DYN Sapphire. The alarms may be disregarded if the control is performing within the assay ranges.

* NOTE: The Assay Value of .001 and Mean Range of ± .001 for NRBC and NRBC/100WBC is entered for the Level L and Level N controls since the instrument will not accept a value of zero. The NRBC concentration for Levels L and N is below the detectable level of the instrument and as such serves as the NRBC negative control. The Level H is the NRBC positive control.

** The mean range does not represent standard deviations (SD).

*** Retic % values for CELL-DYN Ruby are included as separate files on assay disk.

CELL-DYN 29 Plus Control (with Retic)

CONTROL L N H

Exp. 2022-02-11	7 Consecutive Day Open-Vial Stability
-----------------	---------------------------------------

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
CELL-DYN 3700	LOT L13339		LOT N13339		LOT H13339	
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
WOC 10 ⁹ /L	3.2	0.4	7.2	0.7	16.5	2.5
WIC 10 ⁹ /L	3.3	0.5	7.3	1.0	18.6	3.0
WBC 10 ⁹ /L	3.2	0.4	7.2	0.7	16.5	2.5
NEU 10 ⁹ /L	1.8	0.3	4.4	0.8	10.8	2.0
NEU %	56.4	6.0	61.2	6.0	65.6	10.0
LYM 10 ⁹ /L	0.9	0.3	1.7	0.8	3.3	2.0
LYM %	29.2	7.0	24.2	6.0	19.8	10.0
MONO 10 ⁹ /L	0.3	0.2	0.7	0.4	1.6	0.6
MONO %	10.1	5.0	9.9	4.5	9.7	3.0
EOS 10 ⁹ /L	0.1	0.1	0.2	0.2	0.6	0.2
EOS %	3.0	3.0	2.8	2.0	3.3	1.0
BASO 10 ⁹ /L	0.1	0.1	0.3	0.3	0.6	0.6
BASO %	3.0	3.0	3.0	3.0	3.0	3.0
RBC 10 ¹² /L	2.97	0.15	4.33	0.20	5.33	0.28
HGB g/dL	7.8	0.3	12.7	0.5	17.1	0.6
HCT %	24.2	1.5	39.0	2.3	51.0	3.5
MCV fL	81.5	4.0	90.1	4.0	95.7	4.0
MCH pg	26.1	2.0	29.2	2.0	32.1	2.0
MCHC g/dL	32.0	2.3	32.4	3.0	33.5	2.3
RDW %	19.4	2.5	18.3	2.5	16.1	2.5
PLT 10 ⁹ /L	75	20	220	30	480	60
MPV fL	6.9	2.0	6.6	2.0	6.7	2.0
Retic % ¹	5.8	1.5	2.2	1.0	1.0	0.8
IRF ²	0.62	0.38	0.44	0.30	0.39	0.20

Exp. 2022-02-11	7 Consecutive Day Open-Vial Stability
-----------------	---------------------------------------

SYSTEM	CONTROL L		CONTROL N		CONTROL H	
Manual Count ³	LOT L13339		LOT N13339		LOT H13339	
PARAMETER	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **	ASSAY VALUE	± MEAN RANGE **
Retic %	5.6	2.0	2.6	1.5	0.8	0.8

** The mean range does not represent standard deviations (SD).
¹ Retic % values will not load from the Assay Disk. Please enter these values manually.
² IRF is reportable on the CELL-DYN 3700 System, Version 1.1 and higher.
³ Manual values were obtained using the Miller Ocular method.



CELL-DYN, CELL-DYN Sapphire and CELL-DYN Ruby are trademarks of Abbott Laboratories in various jurisdictions.

Abbott Laboratories
 Diagnostics Division
 Abbott Park, IL 60064
 USA

EC REP Abbott GmbH & Co. KG
 Max-Planck-Ring 2
 65205 Wiesbaden
 Germany
 +49-6122-580

MANUFACTURED FOR
 Abbott Laboratories



REF 08H58-01, 08H58-02

9231566B 350491-10 August 2018
 ©2017, 2018 Abbott Laboratories