



| | | | |
|---------|---|---|---|
| CONTROL | L | N | H |
|---------|---|---|---|

IMPORTANT: The barcode is for use only on the CELL-DYN Ruby. Refer to the appropriate System Operator's Manual for proper use of CELL-DYN Calibrator and Control Products.


IMPORTANT: Mixing and Handling

1. Remove a vial of the control from the refrigerator and warm to room temperature (18° to 30° C) for 15 minutes before use.


2. To mix: (**Do NOT mix mechanically or vortex.**)

For a video demonstration, visit www.corelaboratory.abbott and navigate to the Customer Portal → Technical Library → Other Reference Documents → Hematology Aids.

a. Hold the vial vertically and roll each vial between the palms of the hands for 15-20 seconds.

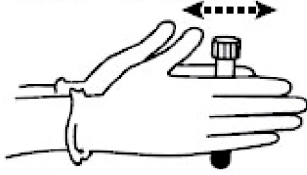

b. Continue to mix by holding the vial by the ends between the thumb and finger, rapidly inverting the vial 20 times end-over-end using a very quick turning motion of the wrist.

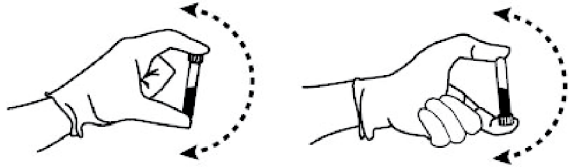

c. Analyze immediately after mixing. Subsequent analyses during this test period may be performed by inverting the vial 5 times prior to instrument analysis.

d. Steps a-c must be repeated upon removing the sample from the refrigerator for the entire open-vial time period regardless of the method of analysis (open tube, cap piercing, auto sample or manual sample).

3. Refer to the appropriate CELL-DYN System Operator's Manual for information about analyzing control specimens.

4. FOR AUTOMATED SAMPLING OR MANUAL CLOSED SAMPLING (CS):

- Refer to the appropriate CELL-DYN Operator's Manual. Remove the vial from the sample handler immediately after sampling.

FOR OPEN-VIAL SAMPLING:

- Aspirate a sample from the vial.
- Carefully wipe the vial rim and cap with a lint-free tissue.
- Replace the cap, ensuring it is on tight.

After sampling, return vial to refrigerator for maximum open-vial stability. If run in the open mode, wipe the threads of both vial and cap before replacing cap and returning to refrigerator.

CELL-DYN 26 Plus Control

ABBOTT
CELL-DYN SYSTEMS



ASSAY SHEET



2024-08-23

8 Consecutive Day Open-Vial Stability

| | | CONTROL L | | CONTROL N | | CONTROL H | |
|-------------------------|------------------------------|-------------|----------------|-------------|----------------|-------------|----------------|
| | | LOT L4162 | | LOT N4162 | | LOT H4162 | |
| SYSTEM | PARAMETER | ASSAY VALUE | ± MEAN RANGE * | ASSAY VALUE | ± MEAN RANGE * | ASSAY VALUE | ± MEAN RANGE * |
| CELL-DYN 3700 SYSTEM | WOC 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | WIC 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | WBC 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | NEU 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | NEU % | N/A | N/A | N/A | N/A | N/A | N/A |
| | LYM 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | LYM % | N/A | N/A | N/A | N/A | N/A | N/A |
| | MONO 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | MONO % | N/A | N/A | N/A | N/A | N/A | N/A |
| | EOS 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | EOS % | N/A | N/A | N/A | N/A | N/A | N/A |
| | BASO 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | BASO % | N/A | N/A | N/A | N/A | N/A | N/A |
| | RBC 10 ¹² /L | N/A | N/A | N/A | N/A | N/A | N/A |
| | HGB g/dL | N/A | N/A | N/A | N/A | N/A | N/A |
| | HCT % | N/A | N/A | N/A | N/A | N/A | N/A |
| MCV fL | N/A | N/A | N/A | N/A | N/A | N/A | |
| MCH pg | N/A | N/A | N/A | N/A | N/A | N/A | |
| MCHC g/dL | N/A | N/A | N/A | N/A | N/A | N/A | |
| RDW % | N/A | N/A | N/A | N/A | N/A | N/A | |
| PLT 10 ⁹ /L | N/A | N/A | N/A | N/A | N/A | N/A | |
| MPV fL | N/A | N/A | N/A | N/A | N/A | N/A | |
| CELL-DYN Ruby SYSTEM | WBC (WOC) 10 ⁹ /L | 4.1 | 0.4 | 6.9 | 0.7 | 16.1 | 2.5 |
| | WBC (NOC) 10 ⁹ /L | 4.2 | 0.4 | 7.1 | 1.0 | 16.3 | 2.5 |
| | NEU 10 ⁹ /L | 2.3 | 0.3 | 4.0 | 0.8 | 9.3 | 2.0 |
| | NEU % | 57.6 | 8.0 | 57.6 | 8.0 | 57.7 | 10.0 |
| | LYM 10 ⁹ /L | 1.0 | 0.3 | 1.8 | 0.8 | 4.2 | 2.0 |
| | LYM % | 25.8 | 9.0 | 26.1 | 9.0 | 26.2 | 10.0 |
| | MONO 10 ⁹ /L | 0.4 | 0.2 | 0.7 | 0.4 | 1.6 | 0.6 |
| | MONO % | 10.4 | 5.0 | 10.0 | 5.0 | 10.0 | 3.0 |
| | EOS 10 ⁹ /L | 0.1 | 0.1 | 0.2 | 0.2 | 0.5 | 0.2 |
| | EOS % | 3.0 | 3.0 | 3.1 | 2.0 | 3.0 | 1.0 |
| | BASO 10 ⁹ /L | 0.1 | 0.1 | 0.2 | 0.2 | 0.6 | 0.6 |
| | BASO % | 3.2 | 3.0 | 3.2 | 3.0 | 3.2 | 3.0 |
| | RBC 10 ¹² /L | 2.88 | 0.15 | 4.16 | 0.20 | 5.31 | 0.30 |
| | HGB g/dL | 7.3 | 0.4 | 11.8 | 0.6 | 15.9 | 0.8 |
| | HCT % | 20.3 | 1.5 | 31.6 | 2.5 | 41.9 | 3.5 |
| | MCV fL | 70.6 | 4.0 | 76.1 | 4.0 | 78.8 | 5.0 |
| MCH pg | 25.5 | 2.0 | 28.4 | 2.0 | 30.0 | 2.0 | |
| MCHC g/dL | 36.2 | 2.3 | 37.4 | 3.0 | 38.0 | 2.3 | |
| RDW % | 13.1 | 2.5 | 11.8 | 2.5 | 12.0 | 2.5 | |
| PLT 10 ⁹ /L | 76 | 20 | 228 | 30 | 540 | 60 | |
| MPV fL | 10.7 | 3.0 | 10.2 | 3.0 | 10.3 | 3.0 | |

* The **MEAN RANGE** does not represent standard deviations (SD).
NOTE: Flags may occur with control materials and should be disregarded.



CELL-DYN and CELL-DYN Ruby are trademarks of Abbott Laboratories in various jurisdictions.

Abbott Laboratories
Diagnostics Division
Abbott Park, IL 60064
USA

EC REP Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

MANUFACTURED FOR
Abbott Laboratories



REF 08H59-01, 08H59-02



9231592B 350004-6 August 2018
©2017,2018 Abbott Laboratories